


# Q·LOGIC 3<sup>e</sup>

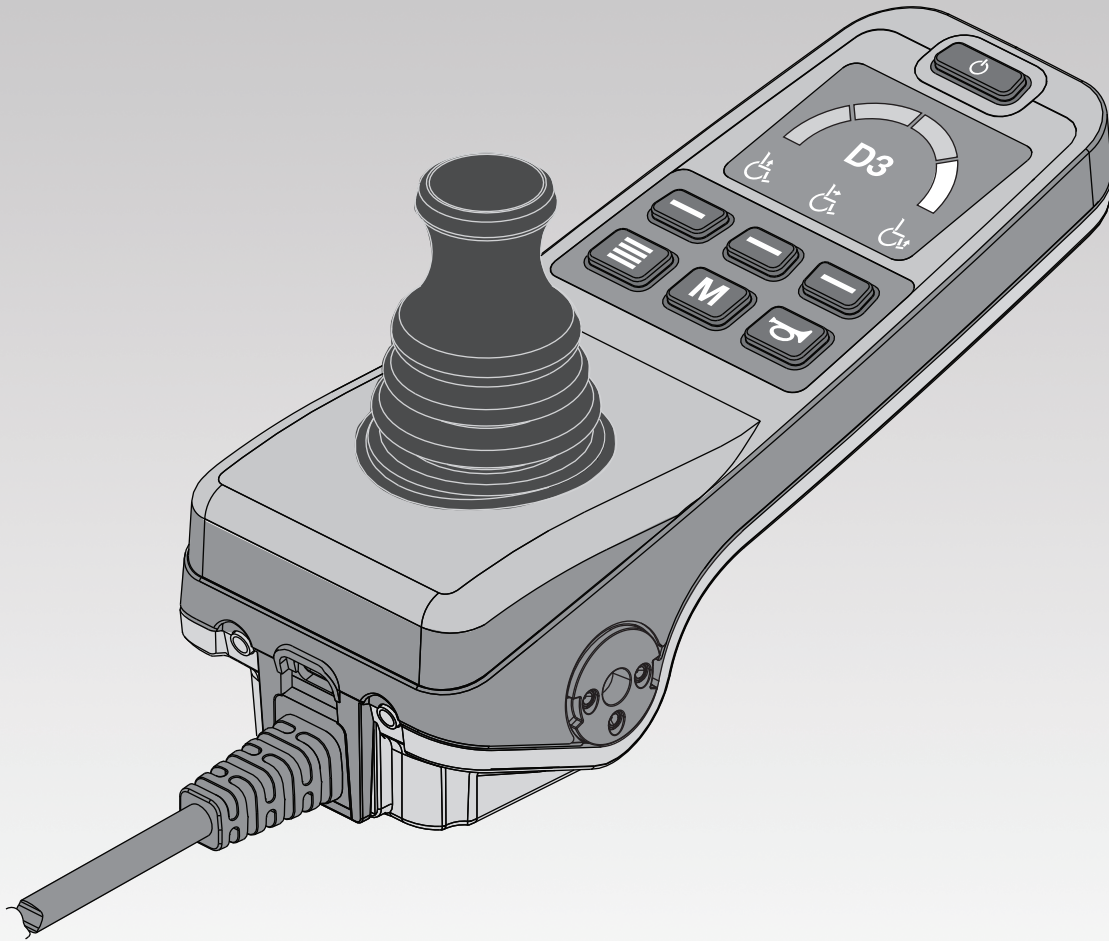


**QUANTUM**<sup>®</sup>

PROVIDING GREATER INDEPENDENCE


# IDENTIFICATION KEY

## LEFT SIDE OF CONTROLLER

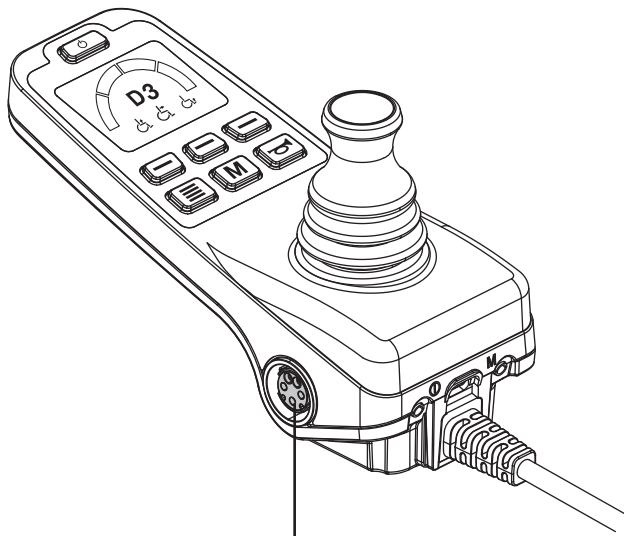

XLR charger port (Left Side)

## RIGHT SIDE OF CONTROLLER

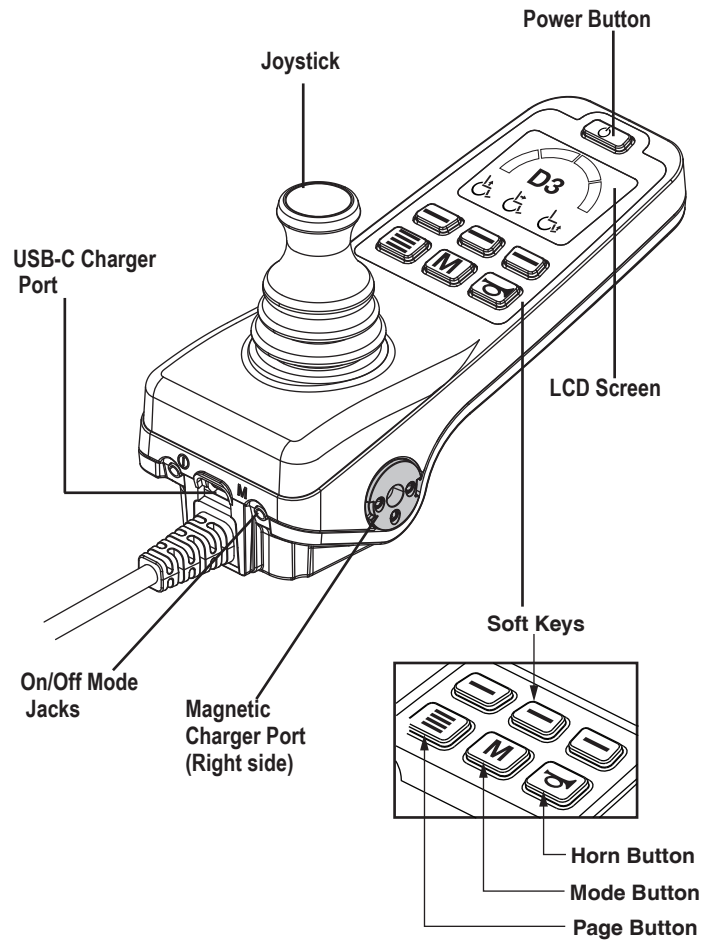

USB-C Charger Port

Joystick

Power Button

LCD Screen

On/Off Mode Jacks

Magnetic Charger Port (Right side)


Soft Keys

Horn Button

Mode Button

Page Button

## BOTTOM OF THE CONTROLLER

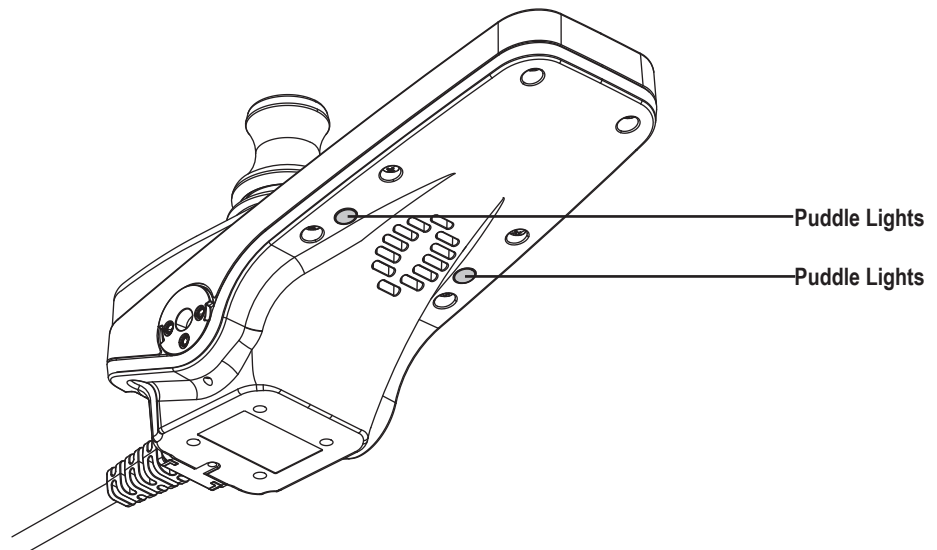


Puddle Lights

Puddle Lights



## Magnetic Charger Adapter

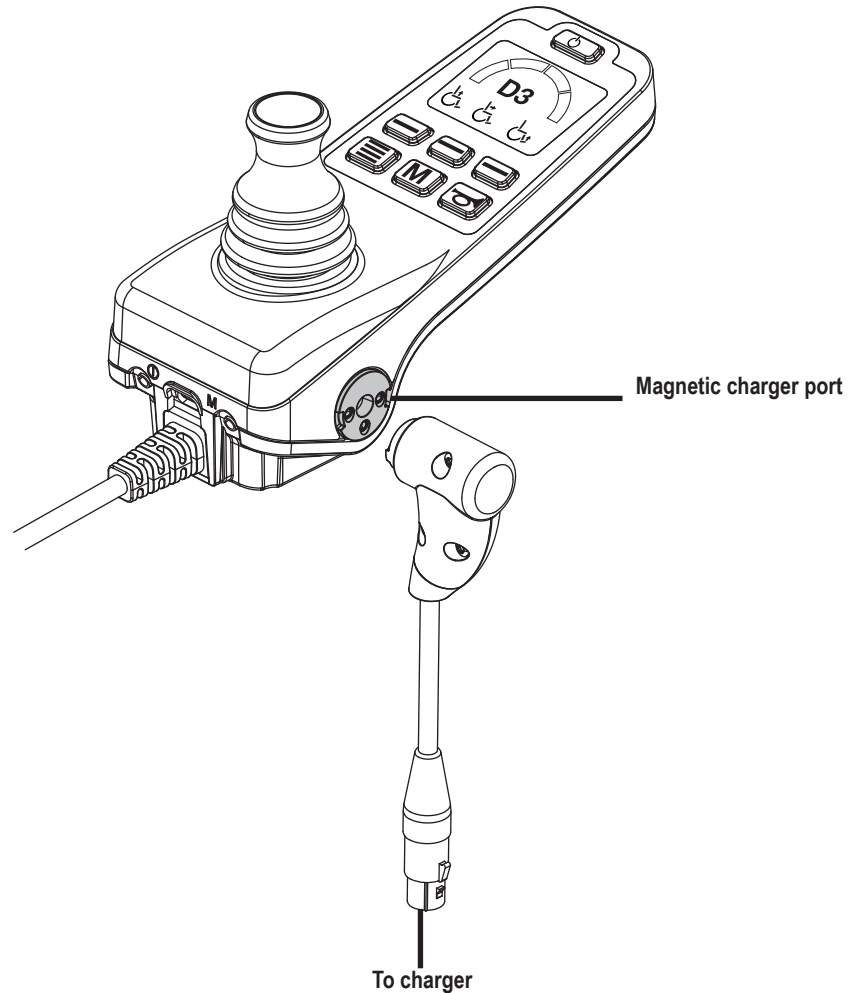


### **WARNING!**

Your controller is equipped with an XLR and magnetic port for charging the power chair. Do not use two chargers at the same time while charging your chair.



Maintain at least 6 in. (15 cm.) separation between magnetic charger connector and any implanted device.

### Attendant Control

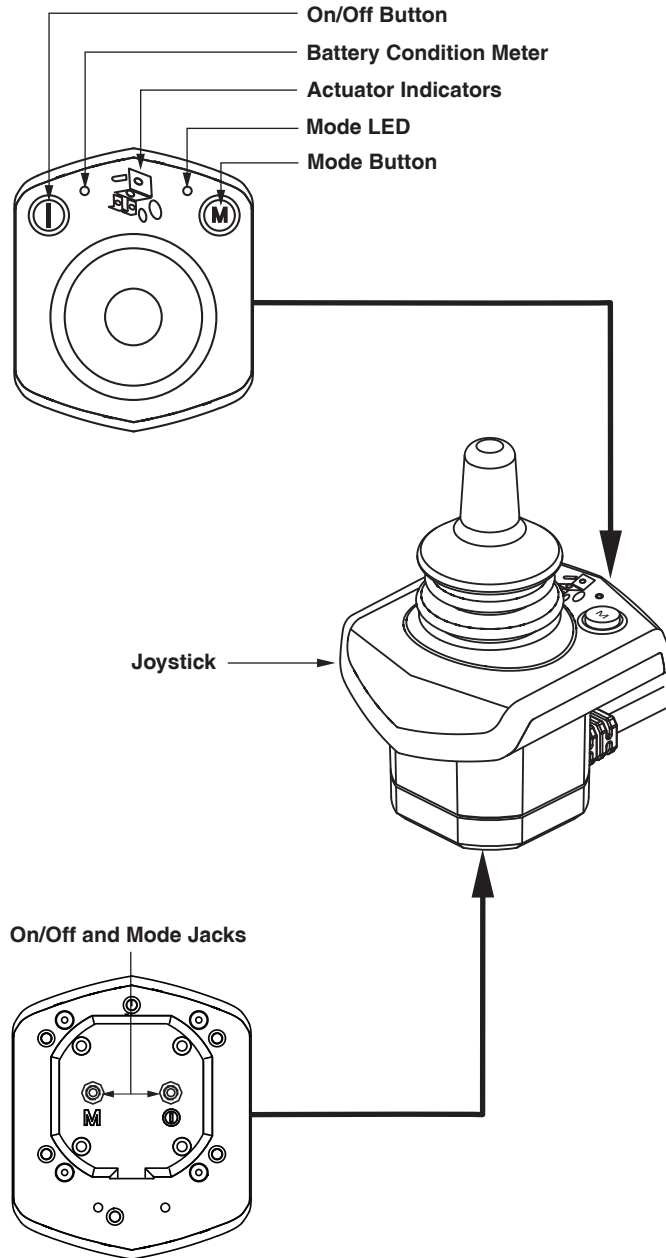



## Safety Guidelines

### WARNING!

A Quantum Rehab Provider or a qualified technician must perform the initial setup of this product and must perform all of the instructions in this manual.

The symbols below are used throughout this owner's manual and on the power chair to identify warnings and important information. It is very important for you to read them and understand them completely.

### WARNING!

Indicates a potentially hazardous condition/situation. Failure to follow designated procedures can cause either personal injury, component damage, or malfunction. On the product, this icon is represented as a black symbol on a yellow triangle with a black border.

### MANDATORY!

These actions should be performed as specified. Failure to perform mandatory actions can cause personal injury and/or equipment damage. On the product, this icon is represented as a white symbol on a blue dot with a white border.

### PROHIBITED!

These actions are prohibited. These actions should not be performed at any time or in any circumstances. Performing a prohibited action can cause personal injury and/or equipment damage. On the product, this icon is represented as a black symbol with a red circle and a red slash.

## Intended Use

A wheelchair component is a device intended for medical purposes that is generally sold as an integral part of a wheelchair, but may also be sold separately as a replacement part.

## Regarding Devices for Prescription Use

Our mobility products and their components are available for sale either as retail ("over-the-counter") or with a prescription. When prescribed, the following statement is applicable.

### WARNING!

**CAUTION!** Federal law restricts this device to sale by or on the order of a physician or other certified personnel licensed by the law of the State (US only) or region in which this personnel practices to use or order the use of the device.

*NOTE: These instructions are compiled from the latest specifications and product information available at the time of publication. We reserve the right to make changes as they become necessary. Any changes to our products may cause slight variations between the illustrations and explanations in this manual and the product you have purchased. The latest/current version of this manual is available on our website.*

*NOTE: This product is compliant with WEEE, RoHS, and REACH directives and requirements.*

*NOTE: This product meets IPX4 classification (IEC 60529).*

*NOTE: The Q-Logic 3e Controller and its components are not made with natural rubber latex. Consult with the manufacturer regarding any after-market accessories.*

**ADVISORY STATEMENT REGARDING CERTAIN COMPONENTRY INCORPORATED BY THIRD-PARTY MANUFACTURERS INTO FINISHED POWER CHAIRS:** When power bases, seating systems, or other components are incorporated into a finished power chair manufactured or assembled by any third party, that third party is responsible to assure the safety, functionality, and legal compliance of the finished power chair. We make no representation concerning the safety, functionality, or legal compliance of the finished power chair or its componentry manufactured by a third party. While we make every effort to assure that our components are distributed responsibly, manufacturers, distributors, and consumers are reminded that finished power chairs must comply with a variety of standards and requirements for governmental safety and functionality.

If it is necessary to physically modify a power chair, including the addition of third-party componentry, to accommodate the medical needs of the power chair occupant, a risk assessment in conformance with ISO 14971, as outlined in ISO\_7176, should be performed.

Changes to power chairs that are likely to affect conformance and risk evaluation include but are not limited to: moving the securement-point brackets; lowering the back-support height; shortening the seat length; adding secondary postural supports that are not firmly attached to the power chair; adding components that have sharp edges (i.e., edges with less than 0.08 in. [2 mm] radius); or any change that compromises the structural integrity of the power chair frame.

## Table of Contents

<b>Product Safety Symbols</b> .....	<b>6</b>
<b>The Q-Logic 3e Controller</b> .....	<b>7</b>
<b>Precautionary Guidelines</b> .....	<b>7</b>
<b>Operating the Q-Logic 3e Controller</b> .....	<b>7</b>
On/Off Button.....	7
Joystick.....	7
Lock/Unlock Procedure .....	7
Speed Adjustment .....	7
Keypad .....	8
Horn Button .....	8
Mode Button .....	8
Page Button.....	8
Soft Keys .....	8
LCD Screen.....	9
Drive Profile.....	9
Drive Profile Selection .....	9
Actuator Adjustment (Seat Screen) .....	10
iLevel® Feature (Optional) .....	11
Elevation (Seat Screen)) .....	11
Tru-Balance 4 Power Anterior Tilt Feature (Optional).....	12
Tilt Seat Screen .....	12
AFP Articualte (Seat Screen).....	13
Memory Seating .....	13
Memory Seat Screen.....	13
User Adjustable Memory positions.....	15
Settings Screen .....	16
USB Charging.....	17
Bluetooth Functions.....	17
Interactive Assist.....	23
<b>Attendant Control</b> .....	<b>24</b>
<b>Thermal Rollback</b> .....	<b>24</b>
<b>Battery Condition Meter</b> .....	<b>24</b>
<b>Battery Life Indicator</b> .....	<b>24</b>
<b>Q-Logic 3e Error Codes</b> .....	<b>24</b>
<b>Care and Maintenance</b> .....	<b>24</b>
<b>Temperature</b> .....	<b>24</b>
<b>Warranty</b> .....	<b>24</b>

## Product Safety Symbols

The symbols below are used on the controller to identify warnings, mandatory actions, and prohibited actions. It is very important for you to read and understand them completely.



Read and follow the instructions in the owner's manual.



Avoid exposure to rain, snow, ice, salt, or standing water whenever possible. Maintain and store in a clean and dry condition.



EMI/RFI - This product has been tested and passed at an immunity level of 20 V/m.



Disposal and recycling - Contact your Quantum Rehab Provider for information on proper disposal and recycling of your Quantum product and its packaging.

## The Q-Logic 3e Controller

The Q-Logic 3e Controller is a fully programmable, modular electronic controller system that allows you to operate your power chair. The hand control will primarily be discussed in this manual along with references to the Attendant Control. Contact your Quantum Rehab Provider for more information.

The controller has been pre-programmed to meet a typical user's needs. The program is adjusted using either a personal computer with software provided by the controller manufacturer or with a handheld programmer, also provided by the controller manufacturer, by your Quantum Rehab Provider or a trained service technician.

### WARNING!

**The controller program can affect speed acceleration, deceleration, dynamic stability, and braking. If it is programmed incorrectly or outside of the safe limits as determined by your healthcare professional, it can create a dangerous situation. Only your Quantum Rehab Provider or a trained technician should program your controller.**

### Precautionary Guidelines

Before operating the Q-Logic 3e Controller please read the following. These guidelines are provided for your benefit and will aid you in the safe operation of the seating system.

- Turn off the power to the controller before you are seated in your power chair.
- Always have assistance when you are being seated in your power chair.
- Follow all of the procedures and heed the warnings as explained in your power chair owner's manual and Consumer Safety Guide.

### Operating the Q-Logic 3e Controller

The Q-Logic 3e hand control is used to operate your power chair and all of its components.

#### On/Off Button

The On/Off button turns the system on and off. It also resets the controller if the LCD screen shows a prompt of a finger pressing the power button when programming has been updated

### WARNING!

**When faced with an emergency situation, release the joystick, then press the on/off to stop the power chair. Use caution. Be advised that pressing the on/off button may cause the power chair to stop abruptly.**

**Always turn the power off when you are stationary to prevent unexpected movement.**

### Joystick

The joystick controls the driving speed and direction of the power chair and is used to navigate the menus on the LCD screen. When the joystick is at rest, it is in the neutral (center) position and the chair is stationary. In order to drive the chair, the joystick must be taken out of neutral. Moving the joystick in any direction will switch the chair from neutral to drive, and the chair will move in the direction indicated by the joystick position. The farther away from the neutral position the joystick is, the faster the chair will move in that direction. To stop chair movement, simply release the joystick or move it back to the neutral position. The chair's electromagnetic brakes will engage after the chair has come to a controlled stop.

### Lock/Unlock Procedure


The Q-Logic 3e Controller comes with a programmable lock/unlock option.

**NOTE: The lock-out feature is not programmed at the factory. To have this feature added, contact your Quantum Rehab Provider.**

#### To lock the controller:

1. Push the On/Off Button once to power on the chair and the controller.
2. Push and hold the Mode Button **M** until the controller turns off. The controller is now locked.

#### To unlock the controller:

1. Push the On/Off Button  once to power on the chair and the controller. The "System Lock Icon" will appear on the display screen.
2. Move the joystick to the full forward position until you hear a beep.

**NOTE: This will take several seconds.**

3. Move the joystick to the full reverse position until you hear a beep.

**NOTE: This will take several seconds.**

4. Release the joystick. The controller is now unlocked.

### Speed Adjustment

The speed adjustment is used to control the speed of the power chair.

#### To change the speed:

1. Push the On/Off Button once to power on the chair and the controller.
2. To increase your speed, press the speed up soft key.
3. To decrease your speed, press speed down soft key.

The speed indicator arc will increase and decrease as the speed arc keys are pressed, providing a visual display of how fast the chair will move.

**Keypad**

The keypad is located directly in front of the joystick. It contains the components that you will use to control your power chair.

**Horn Button** 

The horn button activates a warning horn.



**Mode Button** 

The mode button is used to cycle through Drive Profiles, Seat Mode (if equipped) and Settings Mode.

**Page Button** 

The page button is used to cycle through the soft key pages.

**NOTE:** The soft key pages can be changed by your Quantum Rehab Provider.

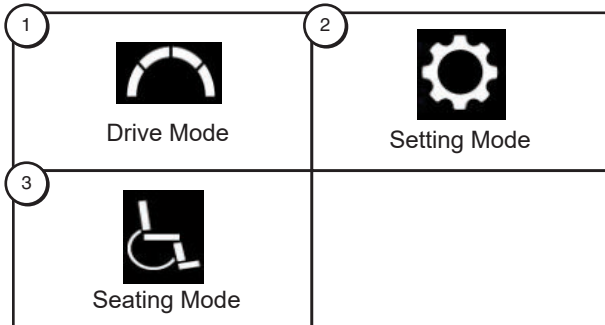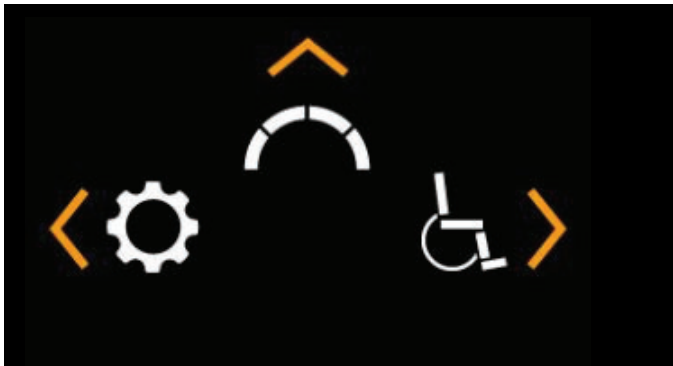**Soft Keys** 

The soft keys will perform the action depicted directly above them. This action may change depending on the screen and it will change with the page button.

**NOTE:** If your Q-Logic 3e Controller is equipped with a lighting system, the soft key buttons will default to a light and left/right turn indicator.

**Light Soft Key** 

The light soft key controls the front headlights and rear running lights.

**To operate the lights:**

1. Push the On/Off Button once to power on the chair and the controller.
2. Press the light soft keys button once to activate the front headlights and rear running lights.
3. Press the light soft key button again to turn off the lighting system.

**NOTE:** On the Drive Screens a light icon will be displayed above the speed arc when the headlights are on.

**Left/Right Turn Indicator Soft Keys**

The left/right turn indicator soft keys toggle either the left or right turn indicators.

**To operate the turn indicators:**  

1. Push the On/Off Button once to power on the chair and the controller.
2. Press the desired turn indicator soft key once to turn on that indicator.
3. Press the same turn indicator soft key again to turn off that indicator.

**NOTE:** If the left turn indicator is activated, pressing the right indicator soft key will turn off the left indicator and activate the right indicator.

**NOTE:** On the Drive Screens a Left/Right turn indicator icon will be displayed next to the speed arc when the Left/Right turn indicator is on.

**NOTE:** The turn indicator soft keys also control the hazard lights. Press both turn indicator soft keys at the same time to activate the hazard lights and press both buttons again to turn off the hazard lights. If the hazard lights are left on and the controller is turned off, the hazard lights will continue to flash. The Q-Logic 3e must be turned back on and both turn indicator buttons must be pressed to deactivate the hazard lights.



**NOTE:** Your Q-Logic 3e controller is equipped with integrated LEDs per path lighting.

**To activate puddle lights:**

1. Press the center soft key to activate the puddle lights. Once the lights are active the gray puddle light icon will turn white.
2. Press the puddle light soft key again to turn off the lights. The puddle lights icon will turn gray to indicate they are off.


**LCD Screen** 

The Q-Logic 3e provides the user with easily intuited feedback information via the LCD screen. The top left corner of the screen will indicate any drive restrictions.

No icon		Full drive speed
Yellow Caution Icon		Limited Drive Speed
Red Warning Icon		Full Drive Lockout

**Outdoor Graphic User Interface** 

The Q-Logic 3e includes an outdoor Graphic User Interface that increases the contrast of the screen to make it easier to be seen in natural light.

To switch from the indoor to outdoor Graphic User Interface or from the outdoor to indoor Graphic User Interface, press and hold page button until the screen changes. 

**Drive Screen**



The LCD screen, provides the current profile, drive operation, and the speed adjustment settings.

Drive/Function Inhibit status is depicted as an icon at the top left of the screen. Actuator adjustment will affect the inhibit status. As your seat and/or back angle increases, a red icon will indicate full drive lockout. If your power chair is equipped with a seating system lift, a yellow icon will indicate speed reduction as the seat is elevated. To return your power chair to full speed, reverse operation.

**To Access the Drive Screen from the Home Screen** 

1. Push the joystick up for the "Drive Screen."

**NOTE:** To access the Drive Screen from any other screen, press the Mode Button  until the desired Drive Profile is reached.

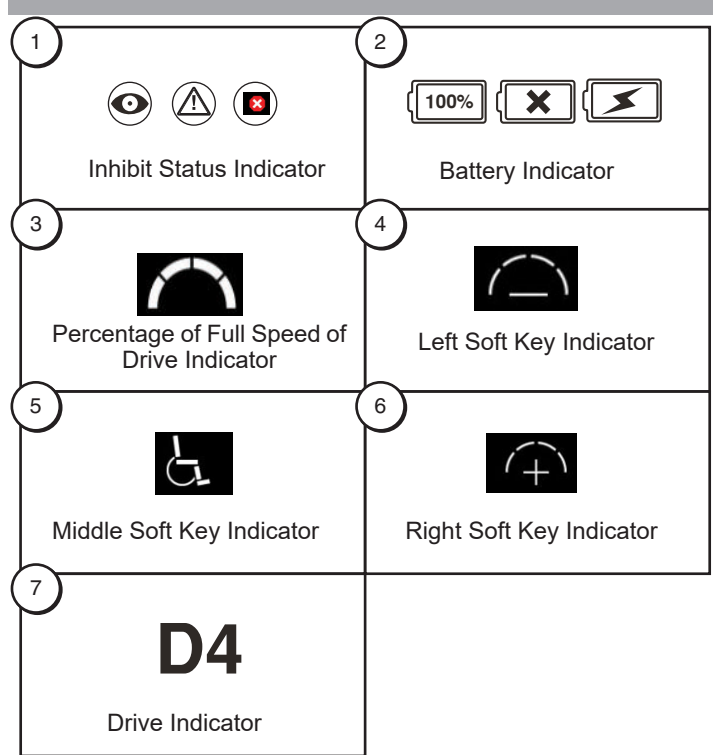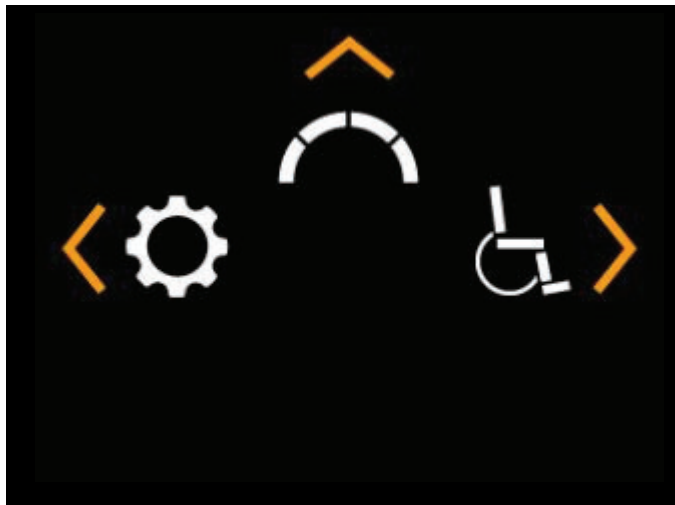

**Drive Profile Selection**

Your Q-Logic 3e Controller may be programmed for up to four drive profiles that allow the system to be custom-tailored to your environment. Profiles 1 – 3 may be selected by pressing the Mode button. The selected profile is displayed on the Drive Screen.

**NOTE:** Drive profiles are set by your Quantum Rehab Provider. Contact your Quantum Rehab Provider to change or add a drive profile.

**To select a profile setting using the Mode Button:**

1. Push the On/Off Button once to power on the chair and the controller.
2. Push the Mode Button to select a drive profile. Cycle through the four available drive profiles.
3. Continuing to push the button will cycle through the Setting screen and Seat screen (if power positioning is present), then back to Drive profile 1.

**! WARNING!**

Drive/Function Inhibit status is read-only in your actuator adjustment (seat screen). This function is disabled in drive mode in order to prevent unintended loss of function in your power chair. Take caution when negotiating, obstacles, ramps, and inclines. Whenever possible, use an attendant in these conditions.

Operating your power chair on inclines and/or with tilt/recline function will affect your power chair's stability.



### Actuator Adjustment (Seat Screen)

The Q-Logic 3e Controller can control five power seat actuators, using the mode button and the joystick.


#### To select and adjust an actuator:

1. Push the On/Off Button once to power on the chair and the controller.
2. Push the Mode Button several times until the seat screen is displayed on the LCD or push the middle soft key button once to go directly to actuator mode.
3. Push the joystick to the left or right to cycle through the available actuators until the desired actuators are illuminated on the actuator indicator. **See figure 1.**
4. When the desired actuator is selected, give a forward command to the joystick to adjust position in one direction or give a reverse command to the joystick to adjust position in the opposite direction. If you continue to push the Mode Button, it will cycle through the Drive screens and Auxiliary screen, and then return to Seat Screen.
5. Push the Mode Button until you return to the desired drive profile.


**NOTE: The Drive/Function Inhibit status is affected only in the actuator adjustment (seat screen).**



1. Tilt Mode

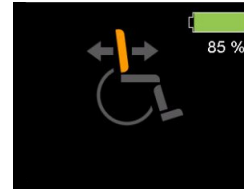


2. Recline Mode

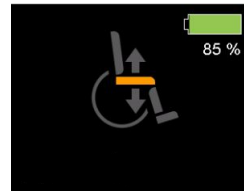


3. Elevating Seat Mode




4. Recline &amp; ELR/ ALR Mode (both)

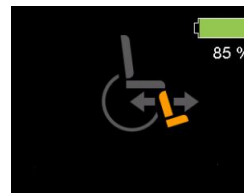

5. Articulating Foot Platform

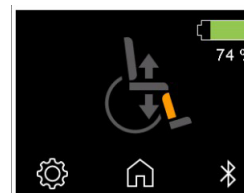

6. \*\*AFP Articulate Seat Screen

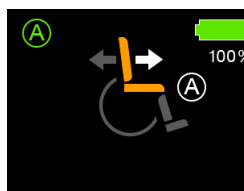

7. \*\*Anterior Tilt Screen



8. \*\*Memory Seat Screen

Figure 1. Actuator Selection

**\*\*NOTE: Only available on seating system with positional feedback.**

### iLevel Feature (Optional)

Your power chair may be equipped with the iLevel feature that will be used in conjunction with your seating system. The iLevel feature is mounted on a Quantum power base for enhanced stability when the seating system is elevated.

There are certain situations, including some medical conditions, where the power chair user will need to practice operating the power chair equipped with iLevel in the presence of a trained attendant. A trained attendant can be defined as a family member or care professional specially trained in assisting a power chair user in various daily living activities.

As you begin using iLevel during daily activities, you will probably encounter situations in which you will need some practice. Simply take your time and you will soon be in full and confident control.

### Elevation (Seat Screen)

The iLevel feature can be activated by selecting the elevating seat actuator.

#### To select and adjust the seat elevation using the Q-Logic 3e Controller:

1. Push the On/Off Button once to power on the chair and the controller.
2. Push the Mode Button several times until the seat screen is displayed on the LCD or push the middle soft key button once to go directly to seat screen. **See figure 1.**
3. Push the joystick to the left or right to cycle through to the elevating seat mode, this is illuminated on the actuator indicator.

**NOTE: If your Q-Logic 3e settings are different than those described, contact your Quantum Rehab Provider. Your Quantum Rehab Provider may have changed the programming of your Q-Logic 3e Controller.**

4. When the elevating seat mode is selected, give a forward command to the joystick to adjust position in one direction or give a reverse command to the joystick to adjust position in the opposite direction.
5. If you want to return to another profile, push the Mode Button to return to the desired drive profile.

#### To activate the iLevel using a single switch:

1. Press once and hold the single switch once to raise the seat and activate iLevel.
2. Press and hold the single switch to lower the seat.

**NOTE: The iLevel feature is automatically engaged when the power elevating seat is activated and the iLevel indicator is displayed on your controller. See figure 2.**

The power chair should be used on an indoor, level surface, such as malls, offices, and/or medical facilities to activate iLevel. Once the seating system actuator is started, the iLevel boots inhibit movement on the front caster arms. The seating system can be reclined or tilted up to 20° while the iLevel feature is activated.

### WARNING!

**The iLevel feature should be used only on a flat, level, indoor surface, such as in malls, offices, and/or medical facilities. Never raise the seat from its lowest position on an inclined surface. Failure to heed this warning can result in the power chair tipping over.**

**The iLevel feature is intended for limited outdoor use on only flat level, hardscape surfaces. If you are unsure of your environment, do not use iLevel outdoors.**

**Never raise the seat into iLevel when operating your power chair on bumpy or uneven surfaces. Failure to heed this warning can result in the power chair tipping over.**

**Always fasten the positioning belt when operating the power chair.**

**Do not operate the iLevel or change the seat height in any way while your power chair is positioned under a fixed object, such as a table or a desk. Keep area clear prior to and during movement.**

### PROHIBITED!

**Never use the iLevel outdoors on uneven terrain (gravel, hills or other uneven, non-compact terrain) or in fields.**

**Never use iLevel while ascending or descending a curb or uneven surface of any height.**

**NOTE: The iLevel feature is equipped with a system that allows the power chair to drive at a predetermined, safe speed when the seat is elevated. Always check to be sure this system is operating properly before using your power chair.**

**NOTE: If a warning or visual indicator appears on your controller stating that iLevel is not engaged, the power chair's speed will be reduced to 25% when elevated. If this warning appears while on level ground, please discontinue use of iLevel feature and consult your Quantum Rehab Provider for assistance.**

## Tru-Balance® 4 Power Anterior Tilt Feature (Optional)

Your power chair may be equipped with the power anterior tilt feature that will be used in conjunction with your seating system. The power anterior tilt feature is mounted on a Quantum power base for enhanced stability when the seating system is tilted forward.

There are certain situations, including some medical conditions, where the power chair user will need to practice operating the power chair equipped with power anterior tilt in the presence of a trained attendant. A trained attendant can be defined as a family member or care professional specially trained in assisting a power chair user in various daily living activities.

As you begin using power anterior tilt during daily activities, you will probably encounter situations in which you will need some practice. Simply take your time and you will soon be in full and confident control.

### Tilt (Seat Screen)

The power anterior tilt feature can be activated by selecting the tilt seat actuator.

#### To select and adjust the seat tilt using the Q-Logic 3e Controller:

1. Push the On/Off Button once to power on the chair and the controller.
2. Push the Mode Button several times until the seat screen is displayed on the LCD or push the middle soft key button once to go directly to seat screen. **See figure 1.**
3. Push the joystick to the left or right to cycle through to the tilt seat mode, this is illuminated on the actuator indicator.

**NOTE: If your Q-Logic 3e settings are different than those described, contact your Quantum Rehab Provider. Your Quantum Rehab Provider may have changed the programming of your Q-Logic 3e Controller.**

1. When the tilt seat mode is selected, give a forward command to the joystick to adjust position in one direction or give a reverse command to the joystick to adjust position in the opposite direction. A forward command, from zero tilt, will result in power anterior tilt mode, on the seat screen. **See figure 1.**
2. When in power anterior tilt mode, the joystick must be returned to neutral and another forward command given to move in power anterior tilt. A reverse command will return to zero tilt.
3. If you want to return to another profile, push the Mode Button until you return to the desired drive profile.

**Note: Your Quantum Rehab Provider may have set an anterior tilt position and a transfer position. If so giving a forward command from zero tilt will move the seat to the anterior tilt position. Releasing the input to neutral and giving another forward command will move the seat to the transfer position.**

#### To activate the Power Anterior Tilt using a single switch:

1. Double press once and hold the single switch once to tilt the seat forward and achieve power anterior tilt.
2. Press and hold the single switch to return the seat to zero tilt.

**NOTE: The power anterior tilt feature is automatically engaged when the power tilt seat is activated in the forward direction and the power anterior tilt indicator is displayed on your controller. See figure 2.**

The power chair should be used on an indoor, level surface, such as malls, offices, and/or medical facilities to activate power anterior tilt. Once the seating system actuator is started, on chairs with iLevel, the iLevel boots inhibit movement on the front caster arms during power anterior tilt. The seating system can be reclined or elevated fully while the power anterior tilt feature is activated.

### WARNING!

**The power anterior tilt feature should be used only on a flat, level, indoor surface, such as in malls, offices, and/or medical facilities. Never tilt the seat forward on an inclined surface. Failure to heed this warning can result in the power chair tipping over.**

**The power anterior tilt feature is intended for limited outdoor use on only flat level, hardscape surfaces. If you are unsure of your environment, do not use power anterior tilt outdoors.**

**Never tilt the seat forward when operating your power chair on bumpy or uneven surfaces. Failure to heed this warning can result in the power chair tipping over.**

**Always fasten the positioning belt and/or other physical restraints such as knee blocks or chest bar, when operating the power chair.**

**Do not operate the power anterior tilt or change the seat height in any way while your power chair is positioned under a fixed object, such as a table or a desk. Keep area clear prior to and during movement.**

### PROHIBITED!

**Never use the power anterior tilt outdoors on uneven terrain (gravel, hills or other uneven, non-compact terrain) or in fields.**

**Never use power anterior tilt while ascending or descending a curb or uneven surface of any height.**



NOTE: The power anterior tilt feature is equipped with a system that allows the power chair to drive at a predetermined, safe speed when the seat is tilted forward. Always check to be sure this system is operating properly before using your power chair.

NOTE: On power chair with iLevel, if a warning or visual indicator appears on your controller stating that iLevel is not engaged, the power chair's speed will be reduced to 0% (full drive lockout) when elevated, during power anterior tilt. If this warning appears while on level ground, please discontinue use of power anterior tilt feature and consult your Quantum Rehab Provider for assistance.

### AFP articulate (Seat Screen)

The footplates can be moved towards the ground, when not in anterior tilt, by selecting the AFP articulate actuator.

#### To select and adjust the AFP articulate using the Q-Logic 3e Controller:

1. Push the On/Off Button once to power on the chair and the controller.
2. Push the Mode Button several times until the seat screen is displayed on the LCD or push the middle soft key button once to go directly to seat screen. **See figure 1.**
3. Push the joystick to the left or right to cycle through to the AFP articulate seat mode, this is illuminated on the actuator indicator.

**NOTE: If your Q-Logic 3e settings are different than those described, contact your Quantum Rehab Provider. Your Quantum Rehab Provider may have changed the programming of your Q-Logic 3e Controller.**

4. When the AFP articulate mode is selected, give a forward command to the joystick to adjust position in one direction or give a reverse command to the joystick to adjust position in the opposite direction. A reverse command will result in footplates moving towards the ground, a forward command will result in the footplates moving away from the ground.

**NOTE: The TRU-Balance 4 system is set in production to stop the footplates before they contact the ground. If the footplates contact the ground, see your Quantum Rehab Provider to get the system reset.**

**NOTE: When the footplates are close to contacting the ground, the system restrict the AFP articulation in the down position, restrict the elevate in the down position, and put the chair into drive lockout.**

### Memory Seating

If your seating system is equipped with memory seating, a saved seat position can be recalled via the Q-Logic 3e electronics.

**NOTE: A memory seat position must be set before use. Consult your Quantum Rehab Provider for assistance.**

The memory seat function is displayed on the Q-Logic 3e seat screen with number 1 through 4. **See figure 1.**

Each memory seat number has a position that can be recalled with a forward command, and another position that can be recalled with a reverse command, for eight (8) possible memory seat positions.

The memory positions can be set to operate in a synched movement (all seat functions move at the same time) or a sequential operation (each seat functions moves, one after the other, in a predetermined order). See your Quantum Rehab Provider for more information.

**NOTE: An anterior tilt position can only be saved to a memory seat position set for synched movement.**

### Memory Seat (Seat Screen)

The memory seat feature can be activated by selecting the memory seat screen.


#### To select and recall the memory seat position using the Q-Logic 3e Controller:





1. Push the On/Off once to power on the chair and the controller.
2. Push the Mode Button several times until the seat screen is displayed on the LCD or push the middle soft key button once to go directly to actuator mode.
3. Push the joystick to the left or right to cycle through to the memory seat mode. This is illuminated on the actuator indicator. **See figure 1.**
4. Push the joystick forward to recall the forward memory seat position.
5. Push the joystick in reverse to recall the reverse memory seat position.

**NOTE: When trying to recall a memory position, if the seat does not move or stops moving before reaching the memory position, an actuator restriction may be active.**

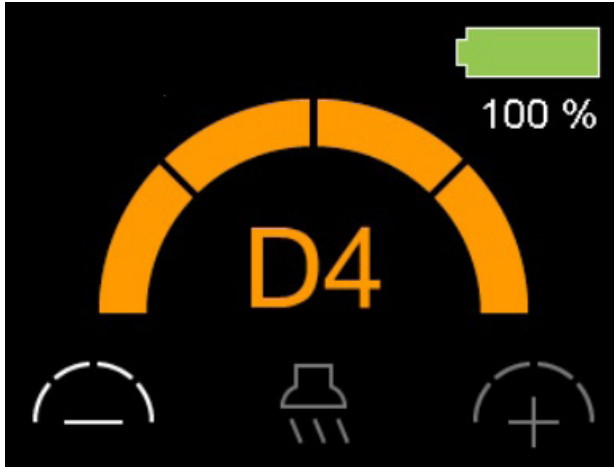Example: If starting with the lift fully elevated and operating a memory position with full posterior tilt (memory position is set for sequential movement), the seat will start to posterior tilt but must stop at 115° as the lift is elevated. Bringing the lift down all the way and performing the memory position again will result in reaching full memory seat position.

**NOTE: An "Error 174 invalid memory position" message is a result of attempting to perform a sequential memory seat position starting from an anterior tilted position. Move seat to zero tilt, or posterior tilt, to perform the sequential memory seat position.**



1a  Safe Elevation	1b  Reduced speed restriction
1c  Full drive lockout	1d  Power Anterior Tilt

**Definitions of Indicators**

- 1a - Safe Elevation
- 1b - Reduced speed restriction
- 1c - Full drive lockout
- 1d - Power Anterior Tilt



iLevel not engaged




Figure 2. Q-Logic 3e Controller Indicators

### User Adjustable Memory Positions

The memory seat numbers 1 through 4 may be designated to be user adjustable. See your Quantum Rehab Provider for more information.

### Saving a user adjustable memory position

1. Move the seat to the position to be saved
2. Push the mode key "M" until the settings icon appears.

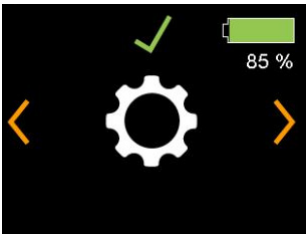

3. Push the joystick right or left to navigate to the "save memory position" screen.



4. Push the joystick forward to enter the memory positions enabled.



5. Push the joystick right or left to navigate to the memory position to be saved.

**NOTE: Each enabled memory position will have 2 programmable positions. One for when the joystick is operated in the reverse position (shown in picture above) and one for operation in forward direction.**



6. While the memory position for programming is on the screen, push the joystick forward to begin the save process.
7. Push the the joystick forward to confirm you would like to save the new memory position. If you would like to cancel this process a reverse command on the joystick will bring you back to step 5.

**NOTE: The disc next to the green checkmark will flash during the saving process.**

8. Once the save process is complete the screen will return to step 3
9. Repeat the steps to save the second position.


**NOTE: The memory positions are customizable. Forward or reverse can be set for whichever position or method of operation that is best for the user.**

**NOTE: If the icon for "save memory positions" is not available when navigating through the menu selections, then no positions have been designated for user adjustable. See your Quantum Rehab provider for more information.**

**Settings Screens** 


The following pages explain the various screens you will use with the Q-Logic 3e Controller. Contact your Quantum Rehab Provider with any questions you may have.


**NOTE: Some screens in this Basic Operating Instructions may differ from your system depending on configuration and options selected.**


**To change the horn volume:** 

1. Push the Mode button, until setting screen is displayed.
2. Push the joystick up to enter the available setting subjects. Push left or right to change the settings.
3. When the desired subject is displayed on the screen, push the joystick up or down to adjust that setting.


After "Horn" is displayed, an up command will increase volume, a down command will decrease volume.

**NOTE: Any time a setting is changed, the system will back up the configuration to the power base. When the system is turned off, "parameter backup"  will be displayed on the screen before the system turns off.**


**To change the horn volume:** 

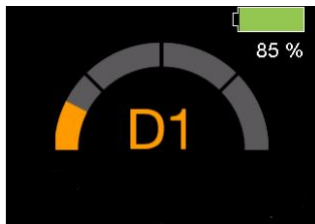

1. Drive Screen

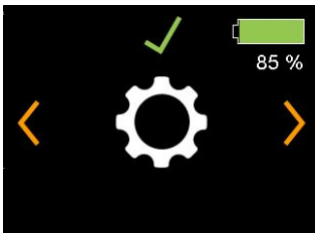

2. Settings

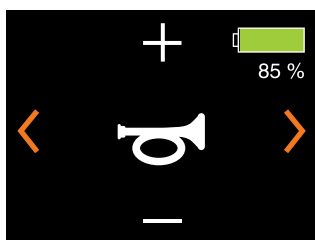


3. Sound

**To change the display brightness:**


1. Push the Mode button, until setting screen is displayed.
2. Push the joystick up to enter the available setting subjects. Push left or right to change settings subject.
3. When the desired subject is displayed on the screen, push the joystick up or down to adjust that subject.

After "Display Brightness" is displayed, an up command will increase brightness and a down command will decrease brightness.


**To change the display setting:**

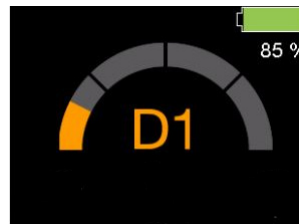

1. Drive Screen

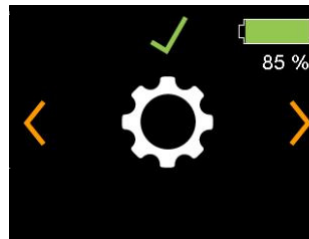

2. Settings

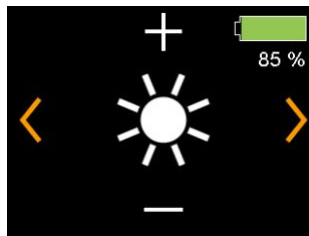

3. Brightness


## Bluetooth

### To activate the Bluetooth function:

The Q-logic 3e joystick is equipped with built in Bluetooth that gives you the ability to connect to computers, smartphones, or tablets. Bluetooth can be paired as a mouse, switch control (iOS), or joystick profile (gaming through PC).

1. Push the On/Off button once to power on the chair and controller.
2. Push the mode button until the settings icon appears.
3. Make a forward command on the joystick to access the Bluetooth Functions menu.
4. Make a right command on the joystick to highlight the Bluetooth function you wish to enable, then push the joystick forward to select that function.
5. Once in the Bluetooth function the input of the chair can be used to navigate through the connected device. Press the mode button to return to the main Bluetooth screen.

### To activate the Bluetooth function:

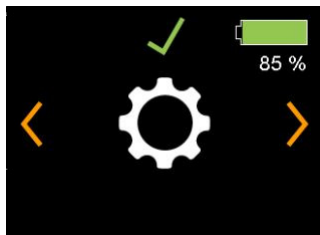

1. Settings



2. Bluetooth Menu

## Mouse Connections

### Pairing a new HID mouse connection function through the bluetooth menu:

1. Push the mode button until the settings menu screen appears.
2. Make a right command on the joystick until the bluetooth function menu appears and push forward on the joystick to enter the bluetooth menu.
3. Make a right command in the bluetooth menu to navigate to the mouse icon and make a forward command to enter the menu.
4. Make a forward command on the joystick to enter the mouse function menu, if no mouse is paired then a mouse icon with "new" underneath will be shown.
5. Make a forward command on the joystick to pair a new device and the joystick bluetooth id will be shown. Find this id in your device bluetooth settings to pair.


### Pairing a new HID mouse connection



1. Settings




2. Bluetooth Menu

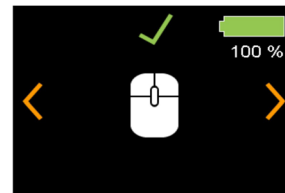

3. Mouse Icon



4. Mouse Function menu

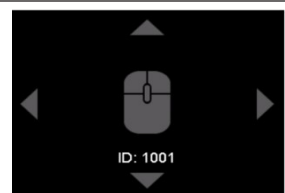

5. Joystick Bluetooth ID

**Using a paired HID mouse connection function through the bluetooth menu**

1. Push the mode button until the settings menu screen appears.
2. Make a right command on the joystick until the bluetooth function menu appears and then make a forward command to enter the bluetooth menus
3. Make a right command in the bluetooth menu to navigate to the mouse icon and make a forward command to enter the menu.
4. Locate the paired mouse you wish to connect to by the device ID located under the mouse icon. Make a forward command to connect to your device.
5. Once the system is connected the mouse icon will turn orange.

**Pairing a new iOS mouse connection function through the bluetooth menu:**

**NOTE: This connection will require assitive touch to be enabled in the accessibility section of the iOS device.**

1. Push the mode button until the settings menu screen appears.
2. Make a right command on the joystick until the bluetooth function menu appears and push forward on the joystick to enter the bluetooth menu.
3. Make a right command in the bluetooth menu to navigate to the mouse icon with iOS and make a forward command to enter the menu.
4. Make a forward command on the joystick to enter the mouse function menu, if no mouse is paired then a mouse icon with "new" underneath will be shown.
5. Make a forward command on the joystick to pair a new device and the joystick bluetooth id will be shown. Find this id in your device bluetooth settings to pair.

**Paired HID mouse connection function through**

**Pairing a new iOS mouse**


1. Settings
2. Bluetooth Menu
3. Mouse icon
4. Device ID
5. Connected Mouse

1. Settings
2. Bluetooth Menu
3. Mouse icon
4. Mouse iOS new
5. Mouse ID new

**Using a paired iOS mouse connection function through the bluetooth menu:**

1. Push the mode button until the settings menu screen appears.
2. Make a right command on the joystick until the bluetooth function menu appears and then make a forward command to enter the bluetooth menus
3. Make a right command in the bluetooth menu to navigate to the mouse icon with iOS and make a forward command to enter the menu.
4. Locate the paired mouse you wish to connect to by the device ID located under the mouse icon. Make a forward command to connect to your device.
5. Once the system is connected the mouse icon will turn orange.


**Paired iOS mouse connection function through bluetooth**

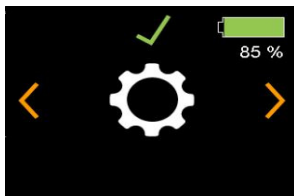


1. Settings




2. Bluetooth Menu

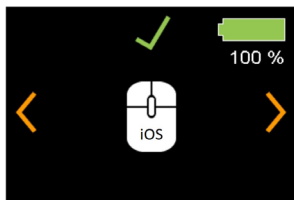

3. Mouse icon

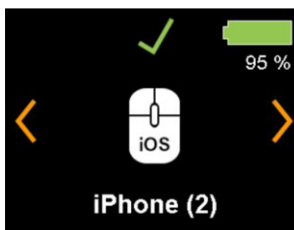

4. Device ID

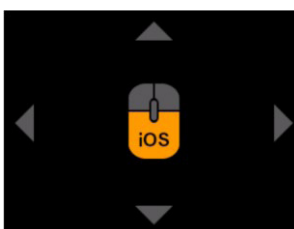

5. Connected mouse

**Pairing a new iOS Switch Control connection function through the bluetooth menu:**

**NOTE: This connection will require Switch Control to be enabled in the accessibility section of the iOS device.**

1. Push the mode button until the settings menu screen appears.
2. Make a right command on the joystick until the bluetooth function menu appears and push forward on the joystick to enter the bluetooth menu.
3. Make a right command in the bluetooth menu to navigate to the Switch Control icon and make a forward command to enter the menu.
4. Make a forward command on the joystick to enter the Switch Control function menu, if no Switch Control is paired then a icon with "new" underneath will be shown.
5. Make a forward command on the joystick to pair a new device and the joystick bluetooth id will be shown. Find this id in your device bluetooth settings to pair.


**Pairing an iOS Switch Control function through Bluetooth**




1. Settings




2. Bluetooth Menu

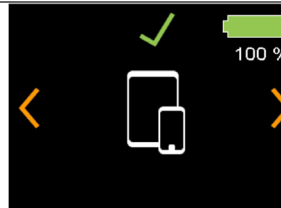

3. Switch Control icon

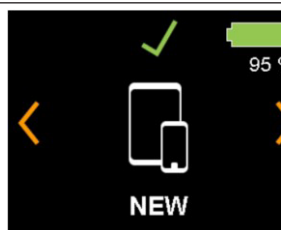

4. Switch Control function menu

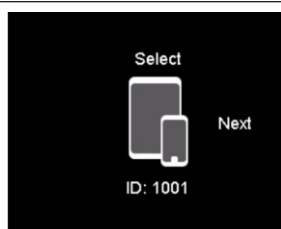

5. Joystick Bluetooth ID




**Using a paired iOS Switch Control connection function through the bluetooth menu:**

1. Push the mode button until the settings menu screen appears.
2. Make a right command on the joystick until the bluetooth function menu appears and then make a forward command to enter the bluetooth menus
3. Make a right command in the bluetooth menu to navigate to the Switch Control icon and make a forward command to enter the menu.
4. Locate the paired Switch Control you wish to connect to by the device ID located under the icon. Make a forward command to connect to your device.
5. Once the system is connected the icon will turn orange.


**iOS Switch Control connection function**




1. Settings

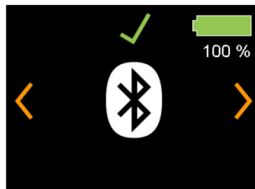


2. Bluetooth Menu

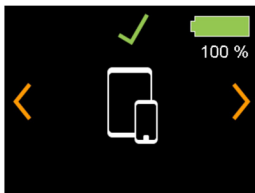

3. Switch Control icon

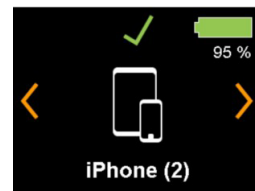

4. Device ID

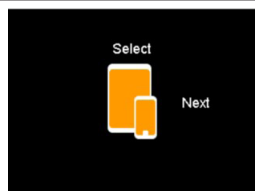

5. Connected Device

**Pairing a new Joystick Profile connection function through the bluetooth menu:**


**NOTE: This connection will require a download of a 3rd party application, Quantum does not support 3rd party applications.**

1. Push the mode button until the settings menu screen appears.
2. Make a right command on the joystick until the bluetooth function menu appears and push forward on the joystick to enter the bluetooth menu.
3. Make a right command in the bluetooth menu to navigate to the Joystick Profile icon and make a forward command to enter the menu.
4. Make a forward command on the joystick to enter the Joystick Profile function menu, if no Joystick Profile is paired then a icon with "new" underneath will be shown.
5. Make a forward command on the joystick to pair a new device and the joystick bluetooth id will be shown. Find this id in your device bluetooth settings to pair.

**Pairing a new Joystick Profile connection function**




1. Settings

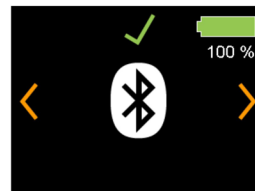


2. Bluetooth Menu



3. Joystick Profile function

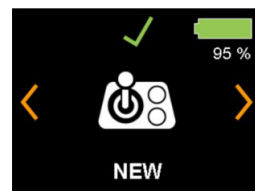

4. Joystick profile function menu

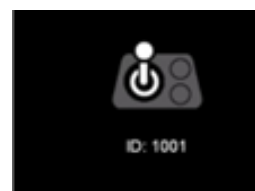





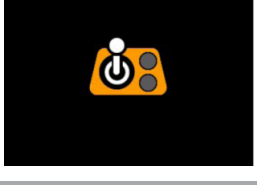
5. Joystick bluetooth ID



**Using a paired Joystick Profile connection function through the bluetooth menu:**

1. Push the mode button until the settings menu screen appears.
2. Make a right command on the joystick until the bluetooth function menu appears and then make a forward command to enter the bluetooth menus
3. Make a right command in the bluetooth menu to navigate to the Joystick Profile icon and make a forward command to enter the menu.
4. Locate the paired Joystick Profile you wish to connect to by the device ID located under the icon. Make a forward command to connect to your device.
5. Once the system is connected the icon will turn orange.

**Using a Paired Joystick Profile**

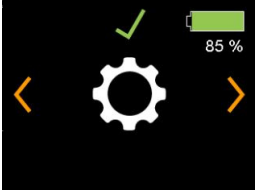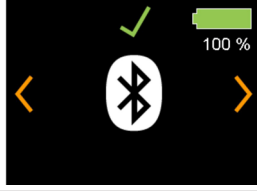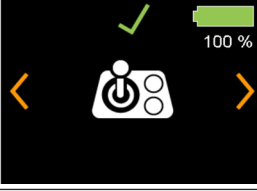	1. Settings
	2. Bluetooth Menu
	3. Joystick Profile function
	4. Device ID
	5. Connected Device






**Mouse Settings**

The mouse settings menu gives you the ability to set how the system operates the clicks for the HID and iOS mouse connections. Clicks can be set as Dwell, through keys, through mode, with joystick commands, and with switches. If dwell is selected as the method for clicks, then a dwell time can also be set.

**Settings mouse clicks or dwell time through the bluetooth menu:**

1. Push the mode button until the settings menu screen appears.
2. Make a right command on the joystick until the bluetooth function menu appears and then make a forward command to enter the bluetooth menus
3. Make a right command in the bluetooth menu to navigate to the Mouse settings icon and make a forward command to enter the menu.
4. Make a forward command on the joystick to enter the Mouse Settings menu.
5. In the clicks menu a forward or reverse command will toggle through the different click options.
6. To set the Dwell time make a right command of the joystick to enter the dwell time menu.

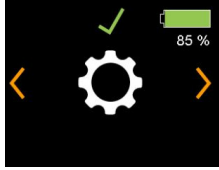**Mouse Settings**

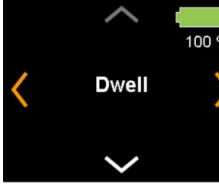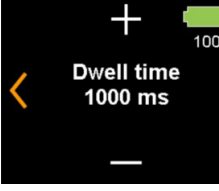	1. Settings
	2. Bluetooth Menu
	3. Mouse Settings
	4. Click menu
	5. Dwell Time


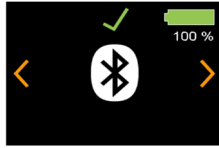

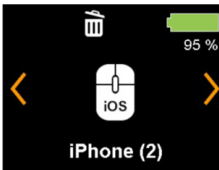


### Delete Bluetooth Connections

Your joystick can be paired with 16 different bluetooth connections. There are 8 available for HID mouse and 8 for iOS devices. When a new device is added to replace a old device, the old bluetooth connections can be deleted.

#### Deleting bluetooth connections:

1. Push the mode button until the settings menu screen appears.
2. Make a right command on the joystick until the bluetooth function menu appears and then make a forward command to enter the bluetooth menus
3. Make a right command in the bluetooth menu to navigate to the Delete connections icon and make a forward command to enter the menu.
4. In the Delete connections menu a left or right command will toggle through the different paired connections.
5. To delete a connection make a forward command on the paired device you wish to remove.
6. Make a forward command to confirm the deletion or a reverse command to cancel.

#### Delete Bluetooth Connections

	1. Settings
	2. Bluetooth Menu
	3. Delete Connections
	4. Paired Connections
	5. Paired Device
	6. Confirm or cancel deletion



**USB Charging Port**

The Q-Logic 3e hand control includes a USB-C port for charging small devices.

The USB port will output up to 1.5 Amps depending on the adaptor.

**The Q-Logic 3e system includes Bluetooth that can be used to program the Q-Logic 3e electronics.**


If a programmer is connected to the power chair, the Q-Logic 3e screen will ask the user to accept Bluetooth access by programming device.

1. To accept the Bluetooth connection, perform a forward command on the power chair's input device.

**Interactive Assist**

The Interactive Assist uses the Q-Logic 3e's Bluetooth connection to allow your provider access to the Q-Logic 3e's Diagnostic information. The connection requires the user to accept access similar to the Bluetooth programming. Refer to the Interactive Assist manual for details.

**To accept Bluetooth access by programming device:**

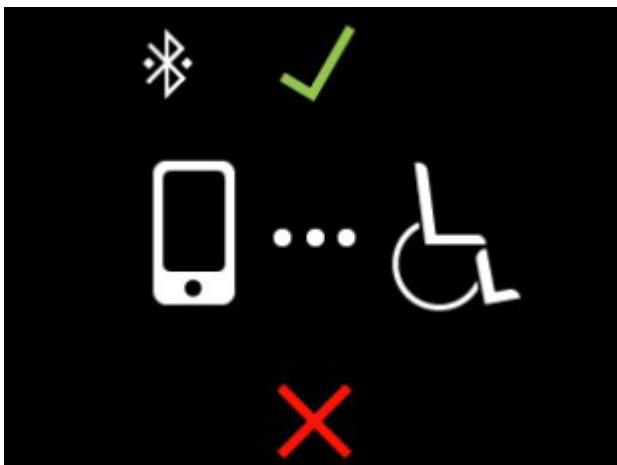

**NOTE: This product has been tested for Electromagnetic and Radio Frequency Interference (EMI/RFI) and has met standard requirements. Please refer to the Consumer Safety Guide for more information regarding EMI/RFI. Although we do not recommend using a cell phone while operating a power chair, the system is capable of receiving and/or making calls through a cell phone device, accessing music files or contact lists, and navigating a device menu.**

**Bluetooth Electronic Specifications**

Bluetooth Output Power	10 mW
Operation Range	40 m LOS (Line of Sight)
Operating Temperature	-40°F to 185°F / -40°C to 85°C
Storage Temperature	-67°F to 302°F / -55°C to 150°C
RF Wireless Technology Type	Co-existence with IEEE 802.11 (AWMA, AFH)
Encryption Security	128-bit

**EMC Testing on Sample Wheelchair**

Susceptibility	Tested to 20V/m to IEC 61000-4-3
Emissions	To CISPR11, Group 1, Class B
ESD	To IEC 610000-4-2

**FCC ID**

FCC Information contains transmitter module

FCC ID:	T7V1316
IC:	216Q-1316

### Attendant Control

For more information on this applications, contact your Quantum Rehab Provider.

The Attendant Control is laid out:


- On/Off Button  
Enables/disables power
- Mode Button  
Cycle through Drive Profiles and Seat Mode (if equipped)
- Battery Condition Meter  
Indicates battery charge
- Actuator LEDs  
Indicates actuators in use
- Mode LED  
Used for profile selection
- Joystick  
They joystick controls speed and direction in Drive mode. Push the joystick to the right to select actuators in Seat mode. Move the joystick forward or backward to activate the desired actuator(s).
- On/Off and Mode Jacks  
Allow for remote On/Off and Mode switch installations

### Thermal Rollback

The Q-Logic 3e Controller is equipped with a thermal rollback circuit that monitors the temperature of the power chair's motors and controller. If either exceeds the safe operating temperature, the controller reduces the output to 20% of full operation level. This reduces the power chair's speed and allows a cool-down period. Once the temperature returns to a safe level, the power chair will resume normal operation.

### Battery Condition Meter


The battery condition meter is in front of the joystick on the controller and as an LED on the Attendant Control.

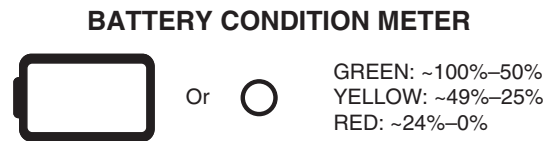

**Figure 3. Battery Condition Meter**

### Battery Life Indicator

The Q-Logic 3e Controller system uses pop-ups to indicate the remaining battery life of the power chair. The system will show information screens when the battery voltage is at 50%, 30%, and 15%. If the pop-ups are not needed, this feature can be disabled, using a programmer, such as Econ-W or Econ-I, under the display settings.

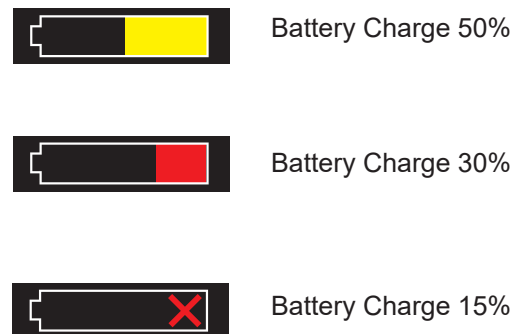




## Q-Logic 3e Error Codes

The Q-Logic 3e Controller displays two types of messages: warning and error. **See figure 4.**

- **Warning:** Alerts you to conditions that may affect power chair functionality and performance.
- **Error:** Alerts you to conditions that influence power chair functionality and performance.

Error codes are displayed on the LCD by number. The following table identifies the error codes that may be easily rectified, probably causes, and possible solutions. If you receive one of these error codes, follow the recommended solution, and if the problem persists, you receive any other error code or experience any other problem with your power chair, contact your Quantum Rehab Provider.

---

	INFORMATION
	WARNING
	ERROR

---

Figure 4. Error Codes

## Care and Maintenance

Refer to your power chair owner's manual for proper cleaning and disposal instructions.

## Temperature













Some of the parts of your power chair are susceptible to extreme changes in temperature. Always keep your power chair between the temperatures of 18°F (-8°C) and 122°F (50°C).

## Warranty





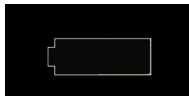



Refer to your power chair owner's manual for specific information on controller warranty.

Error Code	Popup Text	Description (programming device)	Help text (programming device)
1	Motor 1 not connected	Error: Motor 1 not connected	Motor 1 error detected. 1. Turn power off. 2. Inspect motor connections. 3. Turn power on.
2	Motor 2 not connected	Defect: Motor 2 not connected	Motor 2 error detected. 1. Turn power off. 2. Inspect motor connections. 3. Turn power on.
3	Brake 1 not connected	Error: Brake 1 not connected	Brake 1 error detected. 1. Turn power off. 2. Disengage then re-engage brake lever. 3. Inspect motor connections. 4. Turn power off.
4	Brake 2 not connected	Error: Brake 2 not connected	Brake 2 error detected. 1. Turn power off. 2. Disengage then re-engage brake lever. 3. Inspect motor connections. 4. Turn power off.
20	Undervoltage	Error: Undervoltage	Charge Battery.
21	Overvoltage	Error: Overvoltage	Check the battery voltage. Batteries may be overcharged. Avoid driving downhill fast with fully charged batteries.
23	Motor 1 output defect	Error: Motor 1 output defect	Motor 1 error detected. 1. Turn power off. 2. Inspect motor connections. 3. Turn power on.
24	Motor 2 output defect	Error: Motor 2 output defect	Motor 2 error detected. 1. Turn power off. 2. Inspect motor connections. 3. Turn power on.
25	Brake 1 error	Error: Brake 1 error	Brake 1 error detected. 1. Turn power off. 2. Disengage then re-engage brake lever. 3. Inspect motor connections. 4. Turn power on.
26	Brake 2 error	Error: Brake 2 error	Brake 2 error detected. 1. Turn power off. 2. Disengage then re-engage brake lever. 3. Inspect motor connections. 4. Turn power on.
32	Speed reduction (Overtemperature)	Error: Speed reduction (Overtemperature)	Chair speed is reduced to protect system electronics. 1. Turn power off to let system cool down. NOTE: This is the normal system behavior.
34	Joystick not centered	Error: Joystick not centered	Joystick deflected at power up. 1. Turn power off. 2. Release joystick to neutral position. 3. Turn power on.
35	Joystick out of center	Defect: Joystick out of center	Joystick deflected at power up. 1. Turn power off. 2. Release joystick to neutral position. 3. Turn power on.
46	Bluetooth not working	Defect: Bluetooth not working	Bluetooth error detected. Please reboot system. 1. Turn power off. 2. Wait 5 seconds. 3. Turn power on.
47	Suspension lock not engaged - Seek level ground	Defect: Suspension lock not engaged - Seek level ground	Suspension lock not engaged. 1. Seek level ground. 2. Check suspension lock mechanism.
174	Invalid memory position	Error: Memory position invalid	Memory position contains inconsistent data or position cannot be reached  Action 1. Bring system out of anterior tilt and retry memory position 2. Memory position could not be achieved, reset memory position, and retry





Graphic User Interface - General System

Icon Visual	Icon Name	Comments
	Power Cycle	Shown when power cycle is required. Indicates a required action/input.
	DB backup	Shows when system is backing up any changes made.
	DB restore	Shows when system is restoring parameters from Power base.
	Animated bar; Animated element	Shows when system is processing internally.
	Firmware update	Shown if system has detected an update and asks to confirm the installation upon a forward input command. Similarity to BT connection x. Icon Select on top, "Update" text on the bottom. Indicates a required action/input.
	Home drive	A forward command will move to the first drive profile.
	Home seat	A right command will move to the seat screen.
	Home settings	A left command will move to the settings screen.
	Battery outline; Battery X, Battery charging	The battery state shall have the following state indications: 1. Green, 100% segments (total, full) 2. Green, 80% segment 3. Green, 60% segment 4. Yellow, 40% segment 5. Red, 20% segment 6. Red Cross (empty) Percentage is shown below. While charging, an indication shall be added in front of it.
	BT visible	Shows when Bluetooth is enabled.
	BT connected	Shows when Bluetooth is connected.
	Configuring network	Item that indicates that the system modules are being configured. Shows animated bar below.

## GUI Icons - Drive


Icon Visual	Icon Name	Comments
	Drive 1 to 4	Shown while in drive profile. 4 profiles. Example: D1.
	Drive Indoor slow; Drive indoor moderate; Drive outdoor fast; Drive school; Drive work; Drive ramp; Drive sport	Icon to represent drive profile as opposed to Dx, must be programmed by the dealer. Outdoor fast icon shown.
	Drive green circle; Drive purple pentagon; Drive white square; Drive red triangle; Drive blue cross; Drive yellow star	Icon to represent drive profile as opposed to Dx, must be programmed by dealer. Yellow star symbol shown.
	Drive 5 - 100	Shows reachable profile speed with speed adjust setting. 19 items. Example. 60%.
	iLevel	Shown if iLevel drive mode is active while seat is elevated. Shown in drive and seat screens.
	Drive restricted	Shown if a drive speed restriction of any source is active. Shown in drive and seat screens
	Drive Inhibited	Shown if driving is inhibited. Shown in drive and seat screens.
	Attendant Drive 1/2	The attendant drive has 2 profiles.

## Lights, Indicators, Hazard

Icon Visual	Icon Name	Comments
	Light on	Icon shows while light status "on". No icon shows if light status "off."
	Indicator left on	Icon shows while left indicator "on". Shall alternate on and off. No icon shows if left indicator "off."
	Indicator right on	Icon shows while right indicator "on". Will alternate on and off. No icon shows if right indicator "off."
	Hazard on	Icons show while hazard status "on". Will alternate on and off. No icons show if hazard "off." Use indicator icons.



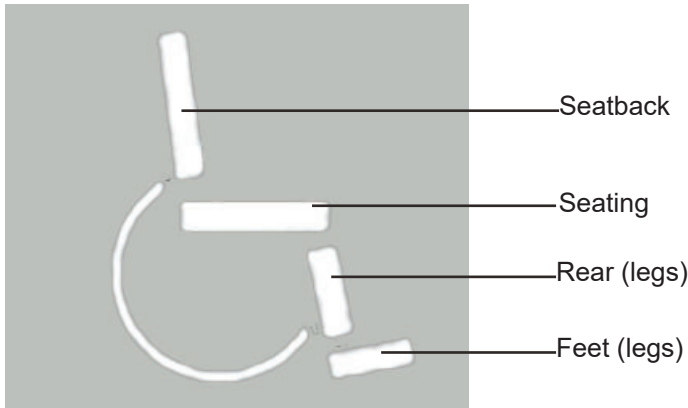

Seat













Seat Icons

Icon Visual	Icon Name	Comments
	Main Seat Tilt Forward	Seating and seatback icon, both highlighted. Direction arrows, forward highlighted.
	Main Seat Tilt Backward	Seating and seatback icon, both highlighted. Direction arrows, backward highlighted.
	Main Seat Recline Forward	Seatback icon highlighted. Direction arrows, forward highlighted.
	Main Seat Recline Backward	Seatback icon highlighted. Direction arrows, backward highlighted.
	Main Seat elevate up	Seating highlighted. Direction arrows, up highlighted.
	Main Seat elevate down	Seating highlighted. Direction arrows, down highlighted.
	Main Seat AFP, Legs out	Used, if both legs operate at the same time: -AFP -Legs -Left leg & right leg combined Foot platform, consisting of 2 elements (1 rear, 1 feet), both highlighted. Arrows, forward highlighted.
	Main Seat AFP, Legs in	Used, if both legs operate at the same time: -AFP -Legs -Left leg & right leg combined Foot platform, consisting of 2 elements (1 rear, 1 feet), both highlighted. Arrows, forward highlighted.
	Main Seat Legs up	Consisting of 2 elements (1 rear, 1 feet) feet highlighted. Left side chair view. Arrows, reverse highlighted (same placement as 4.1)
	Main Seat Legs down	Consisting of 2 elements (1 rear, 1 feet) feet highlighted. Left side chair view. Arrows, reverse highlighted (same placement as 4.2)








## Seat Icons continued

Icon Visual	Icon Name	Comments
	n/a	Seat right leg up. Seat left leg up remains in place
	n/a	Seat right leg down. Seat left leg remains in place.

## Auxillary Functions



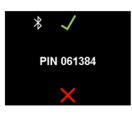








Icon Visual	Icon Name	Comments
	Settings	Settings menu icon, selected with forward input. Left or right input lead to Bluetooth or information menu icon. Shown as well: Select/Next/Previous
	Select	Select menu item. Highlighted if activated by input device forward input.
	Previous	Navigates through menu and settings items. Highlighted if activated by input devices right input (all).
	Next	Navigates through menu and settings items. Highlighted if activated by input device right input (all).
	Change Plus	Highlighted if activated by input device forward input.
	Change Minus	Highlighted if activated by input device reverse input.
	Brightness	Large brightness change icon. Shown as well: Next/Previous/Change plus/minus
	Beep	Large beep volume change icon. Shown as well: Next/Previous/Change plus/minus
	Horn	Large horn volume change icon. Shown as well; Next/Previous/Change Plus/minus.
	BT Man Menu	Main Bluetooth, press up to enable.

Auxillary Functions continued



Icon Visual	Icon Name	Comments
	BT Programming	Wireless Bluetooth programming access icon (tablet/phone).
	BT Connection ID	Shows Power Chair Bluetooth Identification number
	BT Connection	Bluetooth connected push up to abort. Device ID connected on bottom.
	BT Access confirm	Icon shows symbolic chair with a dotted line to the tablet/phone device.
	BT key confirm	Icon shows symbolic key. Shown as well: C2: "BT visible". C3: 'Select' C4: System ID (font size as used for battery) Key number
	Abort	Shown as well: M: 'BT programming' icon T1-T3: Paired device name C2: 'BT connected'
	Provider information	Information menu icon leading to provider information page, shown as well: Select/Next/Previous.

### Soft Key Functions
















The seat soft key icons will correspond to the respective seat screen icons.

Icon Visual	Icon Name	Comments
	Speed Up	Allows to adjust upwards (in steps) max profile; speed down on Drive X (speed arc).
	Speed Down	Allows to adjust downwards adjust (in steps) max profile speed shown on Drive X (speed arc).
	Light Control	Allows light on/off function.
	Indicator Left control	Allows left turn on/off function.
	Indicator Right control	Allows right turn on/off function.
	Home shortcut	Allow access to Home screen.
	Settings shortcut	Allows access to Setting menu.
	BT Shortcut	Allows access to Bluetooth menu.
	Seating Shortcut	Allows access to Seating menu.
	Path Light- white	Allows access to the Path Light. When path light is white, lights are on.
	Path Light - grey	Allows access to the Path Light. When path light is grey, lights are off.

### General exceptions

Icon Visual	Icon Name	Comments
	Warning	Shown on warning as generic icon for most warnings. T2# and warning number. Exceptions for specific user warnings see below.
	Error	Shown on error as generic icon for most errors. T2: # and error number. Exceptions for specific user errors see below.

User exceptions

Icon Visual	Icon Name	Comments
	Information battery yellow	Large battery icons show upon a transition into the following levels. -Green (51%) into yellow (50%) -Yellow (31%) into red (30%) -Red (16%) into red cross Required large icons: -Yellow (50% filled) -Red (30% filled) -Red cross (empty)
	Information battery red	See above
	Information battery empty	See above
	Error battery empty	Error will show an empty battery icon integrated into error icon.
	Error battery charging	Error will show a disconnected power plug icon integrated into error icon.
	Error brakes	Error will show a brake icon integrated into error icon. Drive is inhibited.
	Error overtemperature	Warning will show high temperature integrated into warning icon. Drive at reduced speed with warning icon in C1.
	Warning speed reduction temperature	Warning will show an overcharged battery icon integrated into warning icon. Drive at reduced speed with warning icon in C1.
	Warning speed reduction overvoltage	Warning will show deflected joystick integrated into warning icon.
	Warning joystick deflected	Warning will show deflected joystick integrated into warning icon.
	Warning joystick deflected	Warning will show iLevel icon and a level ground line integrated into warning icon.
	iLevel Warning	Warning will show a traction control (ABS) icon integrated into warning icon for Quantum 4Front.
	Warning traction error	Shows encoder enabled on drive screen.
	Encoder Active	Shows encoder enabled on drive screen.
	Encoder Error	Shows encode error on drive screen.





# Q·LOGIC 3<sup>e</sup>

## **USA**

401 York Avenue  
Duryea, PA 18642  
[www.quantumrehab.com](http://www.quantumrehab.com)

## **Canada**

5096 South Service Road  
Beamsville, Ontario L3J 1V4  
[www.quantumrehab.ca](http://www.quantumrehab.ca)

## **Australia**

20-24 Apollo Drive  
Hallam, Victoria 3803  
[www.quantumrehab.com.au](http://www.quantumrehab.com.au)

## **New Zealand**

3-5/208 Swanson Road  
Henderson, Auckland 0610  
[www.pridemobility.co.nz](http://www.pridemobility.co.nz)

## **UK**

(Authorised UK Representative)  
32 Wedgwood Road  
Bicester, Oxfordshire OX26 4UL  
[www.quantumrehab.co.uk](http://www.quantumrehab.co.uk)

## **The Netherlands**

(Authorised EU Representative)  
De Zwaan 3  
1601 MS Enkhuizen  
[www.quantumrehab.eu](http://www.quantumrehab.eu)  
[www.quantumrehab.nl](http://www.quantumrehab.nl)

## **Switzerland**

(Authorised CH Representative)  
SODIMED SA  
Chemin Praz Devant 12  
1032 Romanel-sur-Lausanne

## **Italy**

Via del Progresso, ang. Via del Lavoro  
Loc. Prato della Corte  
00065 Fiano Romano (RM)  
[www.quantumrehab-italia.it](http://www.quantumrehab-italia.it)

## **France**

26 Rue Monseigneur Ancel  
69800 Saint-Priest  
[www.quantumrehab.fr](http://www.quantumrehab.fr)

## **Spain**

Calle Las Minas Número 67  
Polígono Industrial Urtinsa II, de Alcorcón  
28923 Madrid  
[www.quantumrehab.es](http://www.quantumrehab.es)

## **Germany**

Hövelrieger Str. 28  
33161 Hövelhof  
[www.quantumrehab.de](http://www.quantumrehab.de)

## **China**

Room 508, Building #4  
TianNa Business Zone  
No. 500 Jianyun Road  
Pudong New Area  
Shanghai 201318  
[www.pridechina.cn](http://www.pridechina.cn)



I N F M A N U S 1 5 4